

BioSmart

BIOHAZARD

PREMIUM SPILL KIT

Instructions for use

1. Open the resealable Biohazard Spill Kit satchel and place contents on to a clean surface.
2. Put on all the protective clothing provided: Face Mask & Eye Protection, Protective Apron & Gloves.
3. Open the 250g Bag of BioSmart Premium 'Zeolite' Body Fluid Absorbent Powder. Carefully pour the contents evenly over the spill. Start from the outside and work towards the centre.
4. Ensure spill is generously covered and if required, agitate with scraper provided, until all liquid is absorbed.
5. Remove the spill with the Scoop & Scraper provided, placing the contents into the yellow Clinical Waste Bag. Once complete, add your Scoop & Scraper to the waste bag and seal with a cable tie. Place the sealed Clinical Waste Bag into the second yellow bag provided.
6. Using the BioSmart pH Neutral Detergent and absorbent cloth, clean and remove any residue left behind. Place the used cloth into the second yellow Clinical Waste Bag. Dry the spill area with the second absorbent cloth. Place the second used cloth into the second yellow Clinical Waste Bag.
7. Place Face Mask & Eye Protection, Protective Apron & Gloves into the second yellow Clinical Waste Bag and seal with the Cable Tie provided.
8. Wash your hands with warm soapy water. If soap and water isn't available, use the Antiseptic wipes until a hand washing facility is accessible.
9. You are required to dispose of the Clinical Waste Bag in compliance with your Local Government Regulations.

REMEMBER TO REPLACE YOUR SPILL KIT ONCE USED

REORDER REPLACEMENT KIT: ES-SK-BIO-P OR REFILL (CONTENTS ONLY): ES-SK-BIO-P-R

Call (Int) +61 7 3889 6677 or (Aust) 1300 302 428

Enviro**smart**

www.biosmart.com.au
www.envirosmart.com.au